

PRODUCT TRAINING CARD



Product name

Bindermed

Definition:

Baby Wrist Fixing Band

Field of Use:

It is designed for use in infants undergoing treatment in the neonatal departments. In practice, it is used to fix the baby's hands and ankles to firm places such as incubators, cots and so on. It is used on both hands and ankles.

Function:

The band wraps around the baby's wrists and fixes his hands and feet to certain points, restricts his movements.

Technical Specifications:

There is a velcro* lock on one end of the band, which is used to fasten the band to the wrist after wrapping it around the baby's wrist.

There are connecting ribbons that allow the band to be attached to a place after it is wrapped to the wrist sewn on.

The tape consists of 3 layers laminated** to each other.

The top layer is composed of chardonnayed ***(feathered) fabric. These hairs allow the velcro fixing particles to securely fix the tape.

There is a sponge layer in the middle part that provides softness to the tape. The lowest layer of the band that contacts the skin is made of cotton combed fabric fabric.

The product is disposable and has been sterilized by Gamma Ray.

Velcro*: A special fabric -velcro fabric that is used instead of a snap or button,has micro-level plastic hooks on it and adheres to furry surfaces with these

micro hooks).

Laminating** : (Lamination) The process of gluing different materials together under heat or with the support of adhesive substances using special methods and apparatus and making them into one piece).

Chardoning*** : Combing, combing the surfaces of cotton or synthetic fabrics with a special method the feathering process,if the chardonnayed fabric is feathered, one surface is made soft and velvety or towel-made fabric.

Model

The product is a single model.

Code;

BLH 3001

Measurements;

The tape measures 11 x 3 cm and is 0.4 cm thick.It has 2 connecting cords with a length of 40 cm.

How to use;

The band is wrapped around the baby's wrist.

The velcro locking device at the end is attached to the band and fixed to the wrist. Finally, the fixing cords are connected to the appropriate part of the incubator or cot in order to restrict the movement of the wrist.

Caution:

That the band is securely fixed to the wrist

That the band does not interfere with blood circulation in the wrists

It should be noted that the connecting cords of the tape are thoroughly connected to the fixing point.

Packing;

There is a product in a bag, it is sold in boxes of 25 pieces.