

PRODUCT TRAINING CARD



Product name

Clampmed

Definition:

Urine Control Clamp

Field of Use:

It was developed for the urology service.

It is used to prevent patients from passing urine who cannot control their urine due to prostate disease (unintentionally leaking or dripping) from passing urine. It is designed for adult male patients .

Function:

When placed on the penis, it physically puts pressure on the urinary canal (urethra) with its raised form located in the middle of the part coming under the penis and prevents the flow (passage) of urine.

Technical Specifications:

The clamp consists of a main body that surrounds the penis and an elastic strip glued to this body.



In addition, there is a fixing accessory made of velcro* on the torso that allows it to fix after the clamp is placed on the penis.

The clamp is made of lightweight and flexible polymer (VHPE) **, and polymer foam***** with antibacterial***, anti-fungal****, antiallergic**** properties is glued to it.

The product is multi-use, but it has been sterilized with Gamma Ray for the comfort and trust of the patient during the first use stage and has been put on the market.

Velcro*: A special fabric -velcro fabric that is used instead of a snap or button,has micro-level plastic hooks on it and adheres to furry surfaces with these micro hooks).

VHPE** : It is a kind of polyethylene with very high density.Polyethylene is a polymer (plastic substance) derivative).

Antibacterial*** which does not allow bacteria to live on it). Anti-fungus****: Does not allow fungus to form on it). Anti allergk*****: Does not cause allergies in the body).

Polymer foam:***** Very flexible,soft,polyurethane material.

Model / Measurements

The product is a single model.

It has two size of Normal and Big.

Code;

UCC 01 Normal

UCC 02 Big

How to use;

The clamp is used in everyday life to apply pressure to the urinary canal manually or with another tool without physical intervention.

For this purpose, the tape is wrapped in a sweet firmness to the penis body that is not erect,the part that has a height, the part is on the bottom, that is, on the urethra, and it is fixed with velcro.

*The clamp should be used under the supervision of a doctor.For more information, please read the 'Clempped User Manual and Clempped Patient User Manual").

The clamp is used in everyday life to apply pressure to the urinary canal manually or with another tool without physical intervention.

For this purpose, the tape is wrapped with a sweet firmness to the penis body that is not erect,the part that has a height is partially on the bottom, that is, on the urethra, and fixed with velcro.

Caution:

During the fixing process, care should be taken to have a tightness that will not interfere with blood circulation.

It should not be used while sleeping.

It should not be worn continuously and should be removed at certain periods to ensure urine flow.

Packing;

There is a product in a carton box , it is sold in boxes of 25 small boxes.