

PRODUCT TRAINING CARD



Name: Deckelmed

Description:
Ear Support Bandage

Field of Use:


The product is designed for patients in intensive care and outpatients in remission.

It is used to prevent materials such as pads, tampons, etc., which should be within the ear for medical reasons, from falling, and to support and protect the ear from external factors following surgery.

Function:

Introvert support part of the bandage on the ear covers the ear like a lid, and protects from external factors. It also prevents materials such as pads, tampons, etc. within the ear from falling.

Technical Specifications:

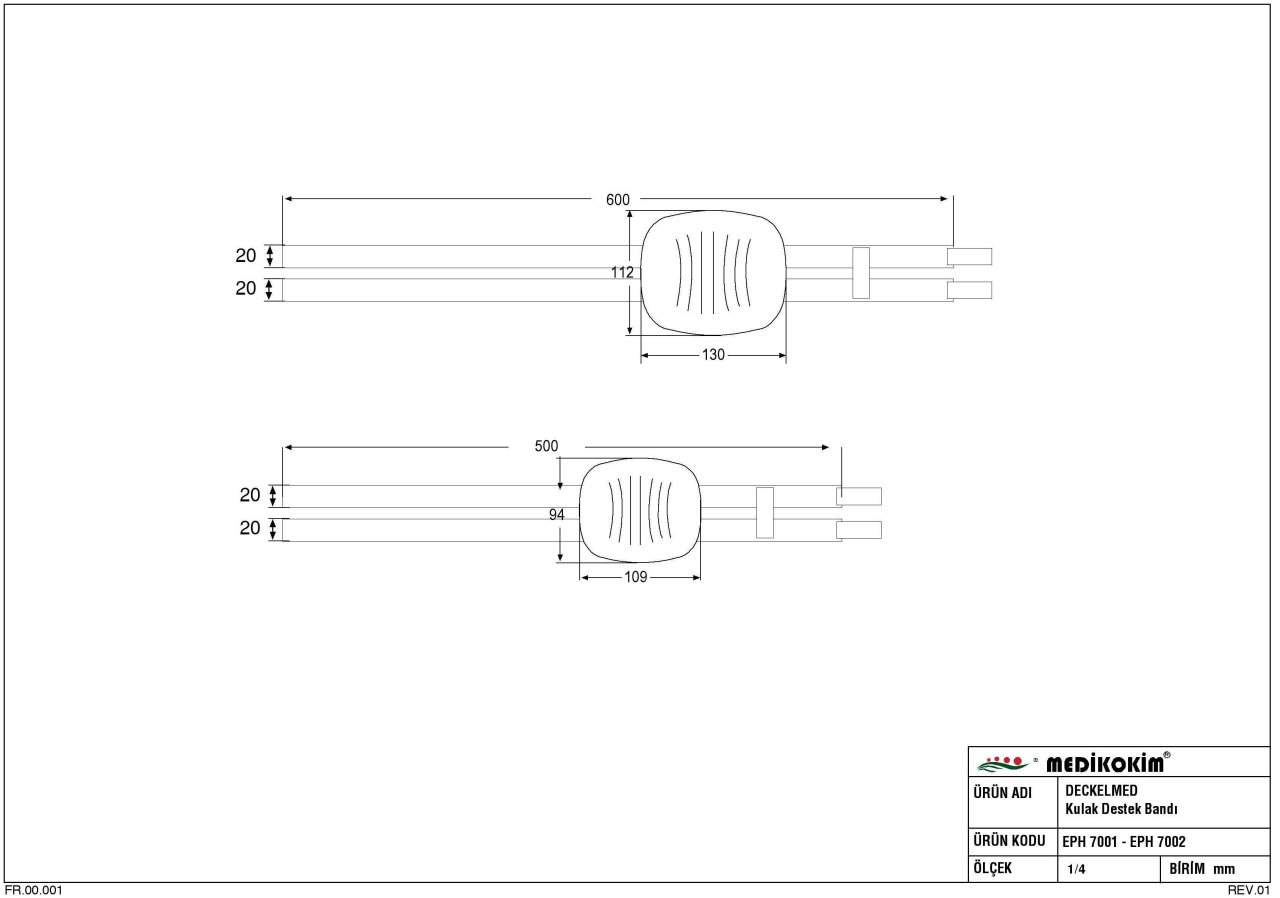

The product is designed to be used for one (single) ear.

It is comprised of a support part, which is placed on the ear to cover and protect it like a lid, and connection apparatus fastening around the head to provide a secure hold.

At each end of the fastening connection apparatus, there are fastening pieces made of Velcro*.

The support part protecting the ear is perforated** so as to allow the ear take the air and also hear.

The bandage is comprised of three layers laminated*** each other.

The top layer is made of napped**** (raised) fabric. These naps ensure that

Velcro fastening pieces fasten the bandages in a secure and safe manner. There is a spongy layer that provides softness to the middle part of the bandage.

The lowest layer of the bandage, which contacts with the skin, is combed cotton.

The product is for single use only, and sterilized with Gamma light.

(**Velcro***: A special kind of fabric used instead of snap fasteners or buttons which contains micro hooks and adhere to fuzzy surfaces by means of said micro hooks).

(**To perforate****: For the textile purposes, a process of drilling on a material or cutting, scoring, slitting it).

(**To laminate*****: (Lamination) A process of attaching different materials using heat or special methods or apparatus with the support of adhesives to make them a single one piece).

(Napping**):** A process for raising the surfaces of combed cotton, cotton or synthetic fabrics through a special method; Napped fabric is a fabric rendered soft and velvety or fuzzy by raising one of its surfaces).

Model / Code / Measurements

It has one model, yet two different sizes for children and adults.



Children's size code: EPH 7002

Adult size code: EPH 7001

Its length is 54 cm for children and 64 cm for adults. It has a width of 0,4 cm.

Use:

Oval shaped, introvert support part of the bandage is placed on the ear so as to fully cover it.

The connection piece at the bottom is moved through the back of the neck towards up from the other side to the upper part of the head, to the forehead.

The connection piece at the top is moved through the forehead, and fixed onto the other connection piece coming from the other side using Velcro adaptors.

Caution:

Connection piece should be fastened well. Attention should be paid to see whether the bandage slips through the head, and to see the support part covers the ear well enough.

Packaging:

Each bag contains one product and is available in boxes of 25.