

## PRODUCT TRAINING CARD



### **Product name**

Fixelmed

### **Definition:**

Urine Bag Holder

### **Field of Use:**

It was developed for the urology service and for male patients who use a bladder in their daily lives.

It is used for safely fixing the urinary bags of patients on the foot (Wandering) to the leg.

### **Function:**

It allows the urine bag to be moved unseen by fixing it to the patient's leg.

### **Technical Specifications:**

It consists of two pieces of elastic, chardonnay \* strip, one narrow and the other wide.



Due to its physical structure (weaving shape), it allows the skin to get air  
In the part of the thick strip that touches the skin, there is a silicone \*\* support reinforcement that prevents the strip from sliding off the leg with weight.

There are 2 Decimeter buttons sewn at certain intervals on the strips. There is also a fixing accessory made of velcro\*\*\* at one end of the strips, which allows them to be fixed after being wrapped to the leg.

It does not contain latex in its structure.\*\*\*\*

The product is non-sterile and can be washed by hand in 30 degree water or in a washing machine with household laundry detergent and reused. No ironing required.

(chardonnay\* : Combing is the process of feathering the surfaces of cotton or synthetic fabrics with a special method, while chardonnayed weaving is a woven product made soft and velvety or terry-like by feathering one surface)

Silicon\*\* : A liquid, resin or polymer (plastic derivative) material with a flexible structure that contains silicon and oxygen atoms in its structure. The type used in the manufacture of medicochemicals was a special type of anti-carcinogenic and anti-allergic structure, which initially had a fluid property and became solid but flexible with heat.

Velcro\*\*\* : A special fabric -velcro fabric that is used instead of snaps or buttons, has micro-level plastic hooks on it and adheres to furry surfaces with these micro hooks.

Latex\*\*\*\*. It is a milky plant extract secreted by many plants found in nature and solidifies when it encounters air. But unless specifically stated, it is the name given to the extract milk obtained from rubber in general and is defined as natural rubber. Products containing latex create an allergy (latex allergy) that causes symptoms in the form of redness, itching, watering and crusting on the skin surface it comes into contact with, this allergy can sometimes reach life hazards.

### **Model**

It is only one model

**Code:**UBH 1001

### **Measurement:**

The thin one with a length of 60 cm is 2 and the thick one is 3 cm wide.

**How to use:**

The tapes are carefully wrapped to the leg one by one and the wide width is on the top, the narrow width is on the bottom, and the velcro locks at the ends are fixed by attaching to the other end of the tape, the chardonnayed surface.

2 Buttons located on both strips - the wider one at the top of the urine bag, the narrower one at the bottom of the urine bag- are passed through the buttonholes (holes).

Thus, the bag is suspended and fixed to the leg.

**Caution:**

During the fixing process, care should be taken to ensure that the tape is tight enough to prevent blood circulation.

**Packing**

There is a product in a bag, it is sold in boxes of 25 pieces.