

## PRODUCT INTRODUCTION CARD



### **Product name**

Fixarmed

### **Definition:**

Urine Catheter Holder

### **Field of Use:**

It was developed for intensive care service.

In male patients using a urinary probe, it is used to fix the probe to the leg so that the part outside the body does not move.

### **Function:**

The product prevents the urine catheter from moving by fixing it on the patient's leg and thus provides comfort to the patient.

### **Technical Specifications:**

It consists of two pieces of elastic, chardonnayed \* strip, one narrow and the other wide.



Due to its physical structure (weaving shape), it allows the skin to get air.

There is a lock system made of velcro\*\*\* on it and it is given a special form to grasp the probe and fix it.

There is also a fixing accessory made of velcro at one end of the band, which allows it to be fixed after being wrapped around the leg.

It does not contain latex in its structure.\*\*\*

The product is non-sterile and can be washed by hand in 30 degree water or in a washing machine with household laundry detergent and reused. No ironing required.

Chardoning\* : Combing was the process of feathering the surfaces of cotton or synthetic fabrics with a special method, while chardonnayed weaving was a woven product made soft and velvety or terry-like by feathering one surface.

Velcro\*\* : A special fabric -velcro fabric that is used instead of snaps or buttons,has micro-level plastic hooks on it and adheres to furry surfaces with these micro hooks.

Latex\*\*\*.It is a milky plant extract secreted by many plants found in nature and solidifies when it encounters air.But unless specifically stated, it is the name given to the extract milk obtained from rubber in general and is defined as natural rubber.Products containing latex create an allergy (latex allergy) that causes symptoms in the form of redness, itching, watering and crusting on the skin surface it comes into contact with,this allergy can sometimes reach life hazards.

### **Model**

It is only one model

**Code:**CH 4001

### **Measurment:**

It is 60 cm lenght and 5 cm width.

### **How to use:**

It is carefully wrapped around the leg and the velcro lock band at the end of the other

it is fixed to the leg by being attached to the tip to the reinforced surface.

The urine probe is placed on the velcro lock on the top of the band, the handles of the lock consisting of two parts are fixed by bringing them over the probe in opposite directions.

### **Caution:**

During the fixing process, care should be taken to ensure that the tape is tight enough to prevent blood circulation.

It should be noted that the velcro lock has a sweet tight grip on the probe.

### **Packing**

There is a product in a bag, it is sold in boxes of 25 pieces.