

# PRODUCT INTRODUCTION CARD



**Product Name:**

Nasomed

**Definition:**

Nasal Cannula Holder

**Field of Use:**

It is designed for use in Intensive Care departments.

It is used to prevent the nasal cannula inserted into the patient's nose from falling out of the nose and to support the cannula.

**Function:**

In the baby model;

The band gets support by gently wrapping the baby's head from the back and fixes the cannula here by passing both prominent ends through the holes of the nasal cannula.

In the children's model;

Elastic rings at the ends of both parts of the band are attached to the child's ears, and the ends pass through the holes of the nasal cannula, fixing the cannula to the nose.

In the adult model;

The velcro lock in the middle part of the tape, it keeps the nasal cannula stable in the patient's nose.

**Technical Specifications:**

The product is produced in 3 different types for infant, child and adult patient use (see Model and dimensions to the title).

The parts of the tape are composed of 3 layers laminated\* to each other. The top layer consists of a chardonnayed \*\*(feathered) fabric. These hairs allow the velcro \*\*\* fixing particles to secure the tape safely.

There is a sponge layer in the middle part that provides softness to the tape.

The lowest layer of the band that comes into contact with the skin is made of

cotton combed fabric. The product is disposable and has been sterilized by Gamma Ray.

(Laminating\*\* : (Lamination ) The process of gluing different materials together under heat or with the support of adhesive substances using special methods and apparatus and making them into one piece). (Charding\*\* : Combing, combing the surfaces of cotton or synthetic fabrics with a special method

the feathering process, while the chardonnayed fabric is a fabric that has been made soft and velvety or terry-like by feathering one surface)

(Velcro\*\*\* : A special fabric -velcro fabric that is used instead of snaps or buttons,has micro-level plastic hooks on it and adheres to furry surfaces with

these micro hooks).

### **Model / Code / Measurements**

There are 3 different models for baby, child and adult.

Baby model ;

The product is designed in one piece and the standard is one size.

it is 43 cm long, its widest place has the appearance of a 6 cm wide band, it is 0.2 cm thick.



Code: NTH 5001

There are fixing particles (accessories) made of velcro at both ends of the band, which serve as fastening after the nasal cannula is inserted.

Child model;

The product consists of two parts that are identical to each other.



Rubber rings suitable for insertion into the ear at one end of the bands are made of velkro, which acts as a fastening after inserting the nasal cannula at the other ends there are fixing particles (accessories) .

Code: NTH 5001 TP

Both pieces are 32 cm long and 0.4cm thick.

Adult model;

The product is one size, it consists of one piece.

There are 4 fixing handles in ribbon appearance and in the shape of the letter X.



There is a fixing accessory made of velcro\* at the cut point of the arms to fix the canon to the nose.

Code: NTH 5002

The width of the band is 1.5 cm, the length of the X-shaped foot is 65 cm from one end of the connecting legs to the other, and the thickness is 0.3 cm.

At each end of the X-shaped arms of the band, there are fixing particles (accessories) made of velcro, which serve to fix the band at the back of the head after the nasal cannula is inserted.

### **How to use;**

In the baby model;

The wide part of the band is brought to the back of the baby's head, the ends of both sides of the face

it is brought from the side to the front and passed through the rings of the nasal cannula attached to this place, and the velcro at the end is attached to the hairy part on the tape and the process is completed.

In the children's model;

The elastic rings of the band are passed through the ear, brought forward from both sides of the face and passed through the rings of the nasal cannula attached to the nose, and the velcro velcro at the ends is attached to the hairy part on the band, completing the process.

In the adult model;

The middle part of the band with a velcro lock is placed on the patient's lip, X-shaped fixing arms are passed through both sides of the face and brought to the back of the head and fixed to each other from the ends with velcro locks. Finally, the velcro lock in the middle part is wrapped to the nasal cannula and the cannula is fixed to the nose

**Caution:**

In the baby model;

Before the tape is passed through the rings of the nasal cannula and fixed, the tension of the tape should be carefully adjusted, it should be noted that it is not loose and that it fixes the cannula thoroughly in place.

In the children's model;

Before the tape is passed through the rings of the nasal cannula and fixed, the band

its tension should be carefully adjusted, care should be taken that it is not loose, does not disturb the ear, and thoroughly fixes the cannula in place.

In the adult model;

It should be noted that the X-shaped fixing arms are well fixed to each other at the back, as well as the velcro accessory located in the middle and fixing the cannula also grips the cannula well

**Packing;**

There is a product in a bag, it is sold in boxes of 25 pieces.