

PRODUCT TRAINING CARD

**Product name**

Portmed

Definition:

Multiple Catheter Holder

Field of Use:

It was developed for intensive care service.

It is used for fixing catheters at the same time in patients who need to connect more than one catheter.

Function:

it provides safety and comfort to the patient by fixing 3 catheters at the same time and deploying them on the patient's arm.

Technical Specifications:

One piece consists of a chardonnay * elastic strip.

Due to its physical structure (weaving shape), it allows the skin to get air. There is a pillow on which catheters are inserted and fixed.

On the pillow, there is a lock system made of velcro ** and which realizes the grasping and fixing of the catheters by giving a special form.

it fixes 3 catheters at the same time.

There is also a fixing accessory made of velcro at one end of the tape, which allows it to be fixed after being wrapped to the arm.

It does not contain latex*** in its structure.

The product is non-sterile and can be washed and reused with household detergent in 30 degree water, by hand or in a washing machine.

Chardoning* : Combing is the process of feathering the surfaces of cotton or synthetic fabrics with a special method, while chardonnayed weaving is a woven product made soft and velvety or terry-like by feathering one surface)

Velcro:** A special fabric -velcro fabric that is used instead of a snap or button,has micro-level plastic hooks on it and adheres to furry surfaces with these micro hooks).

Latex*.**It is a milky plant extract secreted by many plants found in nature and solidifies when it encounters air.But unless specifically stated, it is the name given to the extract milk obtained from rubber in general and is defined as natural rubber.Products containing latex create an allergy (latex allergy) that causes symptoms in the form of redness, itching, watering and crusting on the skin surface it comes into contact with,this allergy can sometimes reach life hazards.

Model

The product is a single model.

Code;

HP MED

Measurements

it is 48 cm long by 5 cm wide.

The fixing cushion is 15 x 8 cm in size

How to use;

It is carefully wrapped around the patient's arm and fixed to the arm by attaching a velcro lock band at one end to the chardonnayed surface of the other end.

The catheters are placed on the velcro locks located on the carrying pad above the tape, the handles of the lock consisting of two parts are secured by bringing them over the probe in opposite directions.

Caution:

During the fixing process, care should be taken to ensure that the tape is tight enough to prevent blood circulation.

It should be noted that velkro locks have a sweet tight grip on the catheters.

Packing;

There are 1 products in a bag, they are offered for sale in boxes of 25 pieces.