

PRODUCT TRAINING CARD



Product name

Restmed

Definition:

Freedom Splint

Field of Use:

It was developed for Intensive Care service.

The cannula probe attached to the face is designed for patients who are trying to remove accessories, etc., or who are prone to self-injury.

Function:

The product allows the patient to move the arm from the shoulder thanks to its strong structure when it is wrapped around the patient's arm - by centering the elbow - but prevents the hand from touching / reaching the face by preventing the elbow from bending.

Technical Specifications:

It has the appearance of a pillow divided into compartments.



Strophor* artillery pieces, which prevent bending of the elbow by showing stiffness, were filled into the compartments.

There is a lock system consisting of velcro** bands on one short edge of the armband that allows it to be fixed when wrapped around the arm.

There are 2 plastic clips on the long sides of the armband that allow it to be attached to the patient's clothing to prevent it from slipping after wrapping the arm.

The surface in contact with the patient is a cotton fabric.

The product is non-sterile and can be washed by hand in 30 degree water or in a washing machine with household laundry detergent and reused.

No ironing required.

Styrofoam *: Styrofoam is a closed porous, usually white colored thermoplastic synthetic material obtained by rolling from the oil part of crude oil, in the class of plastics, in the form of foam, obtained by rolling

Velcro **: A special fabric - velcro fabric that is used instead of a snap or button, has micro-level plastic hooks on it and adheres to furry surfaces with these micro hooks).

Model

It is produced in two models Standart and Soft model.

Standart Model



The outer surface, which does not touch the patient, is made of waterproof PVC material. The inner surface in contact with the patient is made of cotton fabric.

Soft Model



The outer surface, which does not come into contact with the patient, is made of air-permeable and wipeable material. The inner surface that comes into contact with the patient is made of cotton fabric.

Code and Measurements

Model	Ebad	Adet/Poşet	Poşet/Kutu	Kutu Ebadı	Kutu Ağırlığı	Sipariş Kodu	UBB Kodu
Standart XS	23 x 34 cm.	1	10	25 x 50 x35 cm.	1000 gr.	RSFRSW 01	8680508102912
Standart S	28 x 37 cm.	1	10	25 x 50 x35 cm.	1500 gr.	RSFRSW 02	8680508102929
Standart M	30 x 47 cm.	1	10	25 x 50 x35 cm.	2250 gr.	RSFRSW 03	8680508102936
Standart L	36 x 64 cm.	1	10	25 x 50 x35 cm.	4000 gr.	RSFRSW 04	8680508102943
Standart XL	49 x 69 cm.	1	10	25 x 50 x35 cm.	5000 gr.	RSFRSW 05	8680508102950
Soft XS	23 x 34 cm.	1	10	25 x 50 x35 cm.	1000 gr.	RSFSSW 01	8680508102967
Soft S	28 x 37 cm.	1	10	25 x 50 x35 cm.	1500 gr.	RSFSSW 02	8680508102974
Soft M	30 x 47 cm.	1	10	25 x 50 x35 cm.	2250 gr.	RSFSSW 03	8680508102981
Soft L	36 x 64 cm.	1	10	25 x 50 x35 cm.	4000 gr.	RSFSSW 04	8680508102998
Soft XL	49 x 69 cm.	1	10	25 x 50 x35 cm.	5000 gr.	RSFSSW 05	8680508103001

How to use;

The armband -with the elbow centered- is wrapped around the patient's arm . It is fixed to the arm thanks to Velcro bands.

It is connected to the patient's clothing by means of a plastic clip located at both ends.

Caution:

During the fixing process, it should be noted that the armband should be tight enough to prevent blood circulation.

Packing; There is one product in a bag, it is offered for sale in boxes of 10 pieces.