



# USAGE INSTRUCTIONS

## **BAREMED**<sup>®</sup> URINE CONTROL BAND


MODEL : STANDARD  
REF. NO : UCB 6001

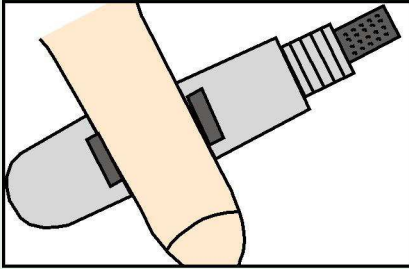


*Place the silicone support under the penis*

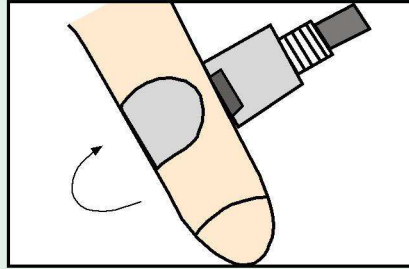


*Wrap the oval tip around the penis*

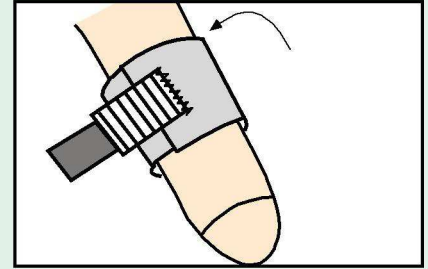


*Put the straight end onto the oval tip*

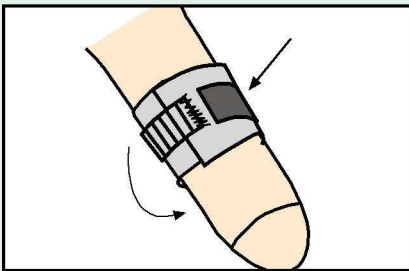

*Stretching the elastic strap close the velcro. Do not overtighten*

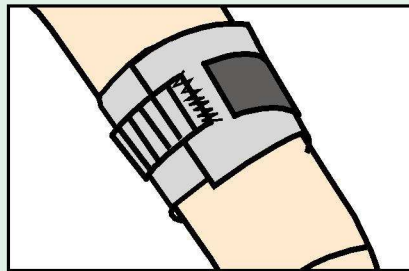

*Pay attention for not to hinder the blood circulation*

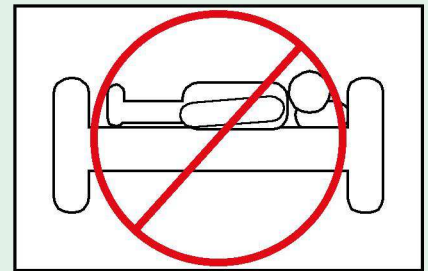

*Do not use during sleep*




*Bant prevents urinating and dripping  
Do not accumulate your urine using band  
When feeling, discharge your urine system at once*



▷ **STERILE**

▷ Do not resterilize

▷  Single use

▷ Do not use opened and damaged packs