



USAGE INSTRUCTIONS

CLAMPMED[®] URINE CONTROL CLAMP


MODEL : NORMAL - BIG
REF. NO : UCC 01 - UCC 02

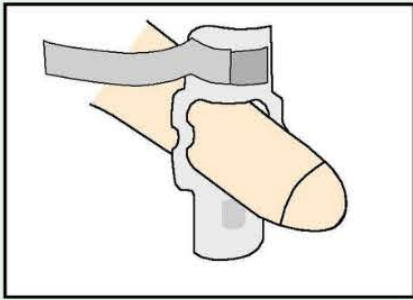


*Slip the clamp on the penis.
Keep the strap above*

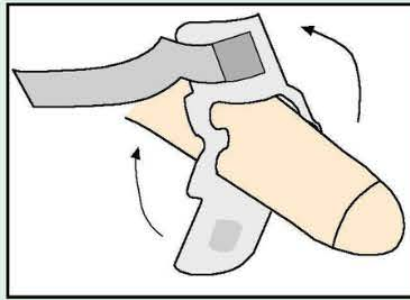


*Fold the clamp.
Do not overtighten*

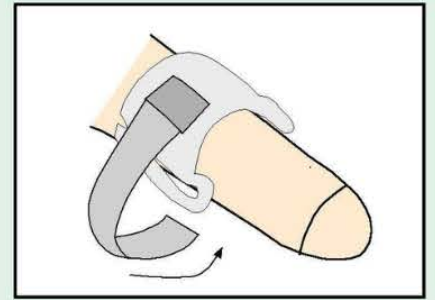

*Wrap the ribbon around
and secure*

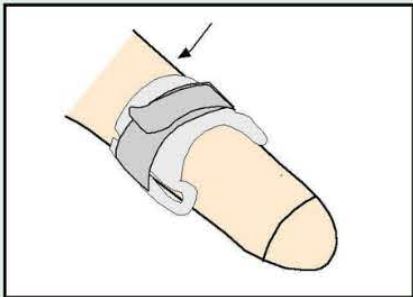

*Pay attention for not to hinder
the blood circulation*

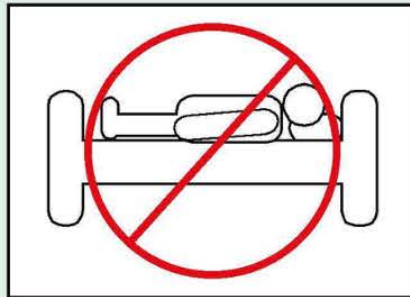

Do not use during sleep



*Band prevents urinating and
dripping Do not accumulate your
urine using band. When feeling
discharge your urine system
at once*

Use Klomp always under medical supervision. Klomp is only for urinary incontinence and to prevent dripping. Before using it detect the urination period of your body and urinate in these periods minimum.



▷ Non-steril

▷ Reusable

▷ Do not use opened and damaged packs