



USAGE INSTRUCTIONS

FESTERMED[®] LIMP HOLDER FOR ADULTS


MODEL : FOOT
REF. NO : ALH 2002

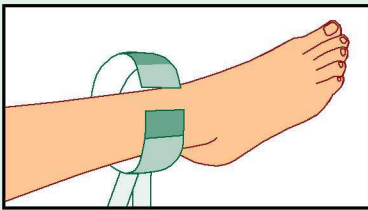


Wrap the band around the wrist

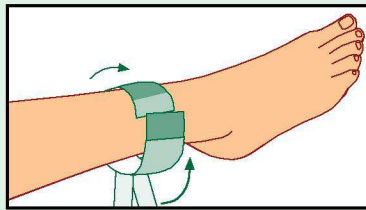


Pay attention for not to hinder the blood circulation

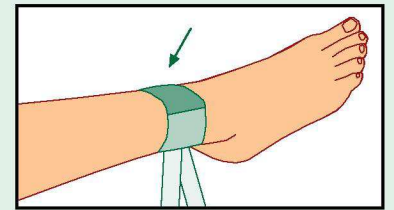

Pay attention for not to hinder the blood circulation

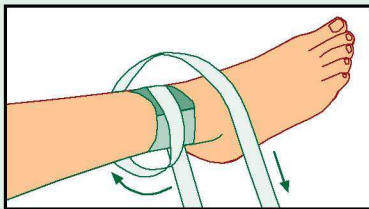

If necessary wrap the ribbons once more time around the leg

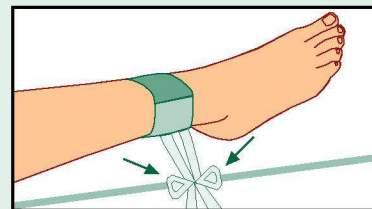

Connect the ribbons to a fixed point



▷ **STERILE**

▷ **Single use**


▷ *Do not resterilize*

▷ *Do not use opened and damaged packs*


USAGE INSTRUCTIONS

FESTERMED[®] LIMP HOLDER FOR ADULTS


MODEL : LEG
REF. NO : ALH 2003

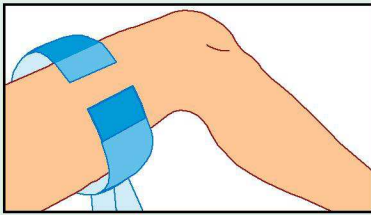


Wrap the band around upperleg

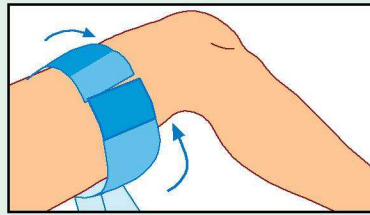


Pay attention for not to hinder the blood circulation

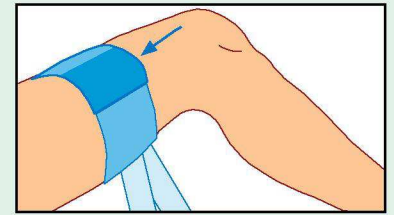

Pay attention for not to hinder the blood circulation

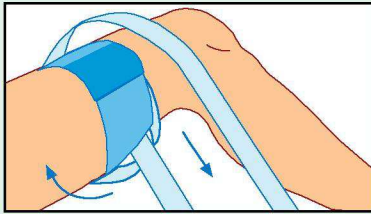

If necessary wrap the ribbons once more time around the leg

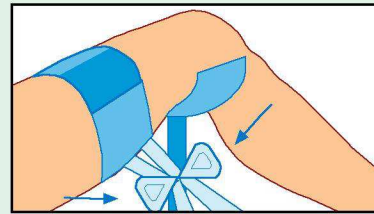

Connect the ribbons to a fixed point



▷ **STERILE**

▷ **ⓧ** Single use

▷ Do not resterilize

▷ Do not use opened and damaged packs


USAGE INSTRUCTIONS

FESTERMED DELUX®


LIMB HOLDER FOR ADULTS

MODEL : FOOT


REF. NO : ALH 2007

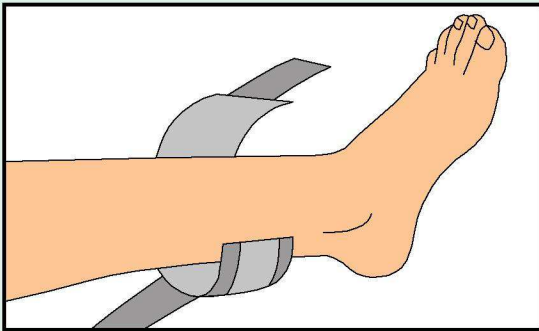


Wrap the band around the wrist

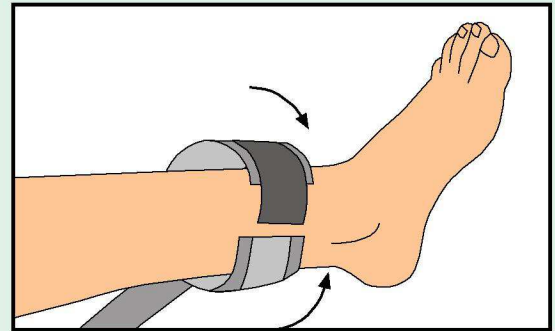

Pay attention for not to hinder the blood circulation

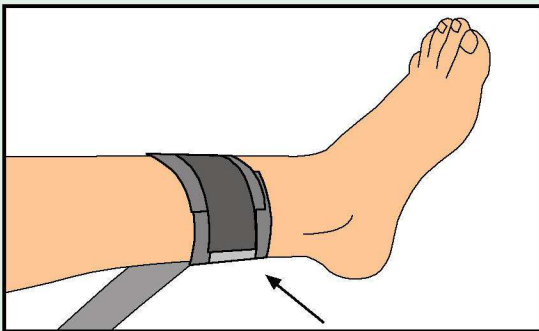

Close the velcro

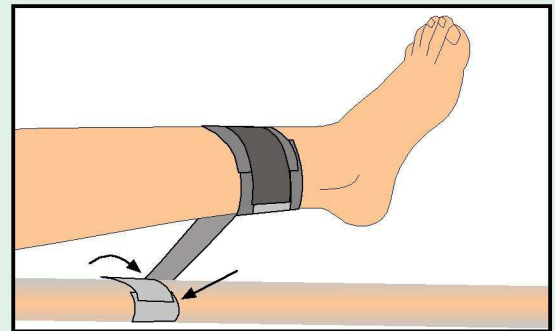

Connect the ribbon to a fixed point



▷ **STERILE**

▷ **Single use**

▷ *Do not resterilize*

▷ *Do not use opened and damaged packs*


USAGE INSTRUCTIONS

FESTERMED DELUX[®]


LIMB HOLDER FOR ADULTS

MODEL : HAND


REF. NO : ALH 2006

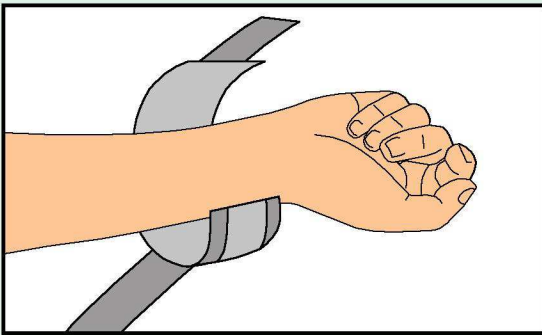


Wrap the band around the wrist

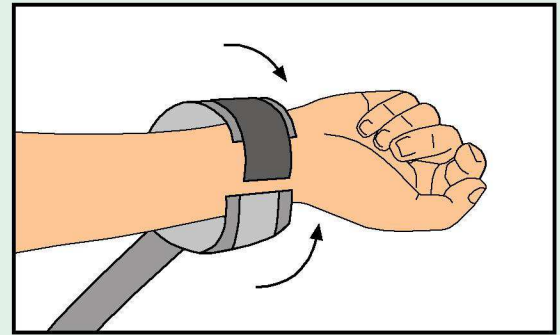

Pay attention for not to hinder the blood circulation

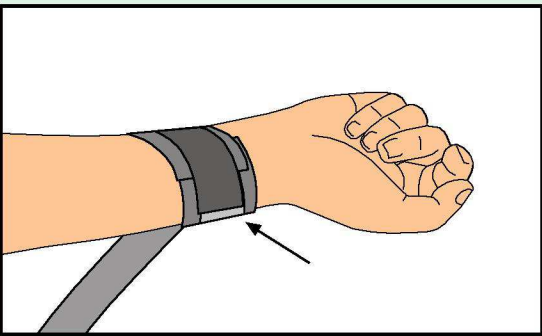

Close the velcro

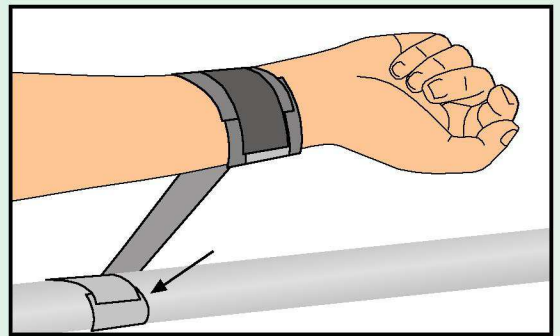

Connect the ribbon to a fixed point



▷ **STERILE**

▷ **Single use**


▷ *Do not resterilize*

▷ *Do not use opened and damaged packs*


USAGE INSTRUCTIONS

FESTERMED[®] **LIMP HOLDER FOR ADULTS**


MODEL : HAND
REF. NO : ALH 2001

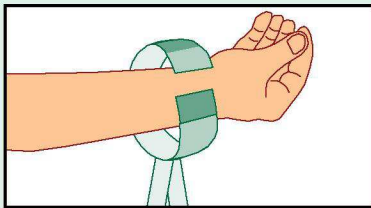


Wrap the band around the wrist

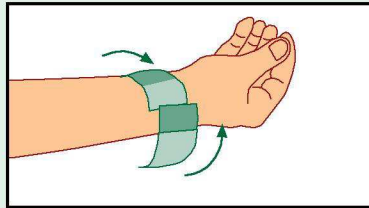


Pay attention for not to hinder the blood circulation

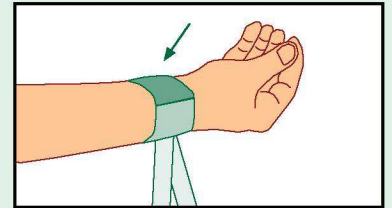

Pay attention for not to hinder the blood circulation

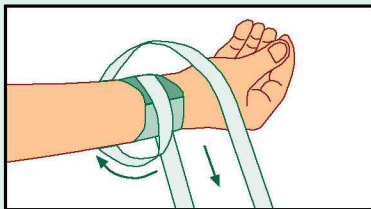

If necessary wrap the ribbons once more time around the leg

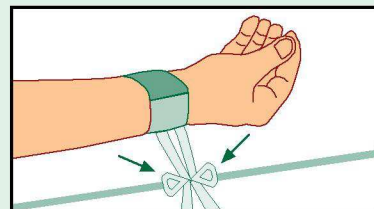

Connect the ribbons to a fixed point



▷ **STERILE**

▷ **Single use**

▷ *Do not resterilize*


▷ *Do not use opened and damaged packs*

USAGE INSTRUCTIONS


FESTERMED METAL[®]

LIMB HOLDER FOR ADULTS


MODEL : FOOT
REF. NO : ALH 2011

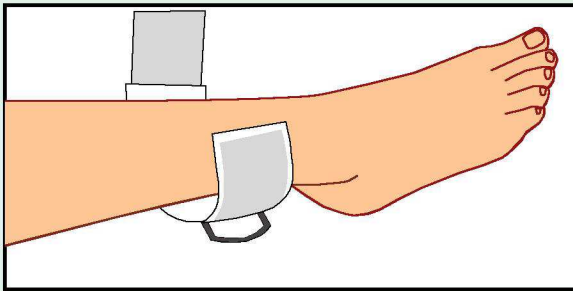


Wrap the band around ankle / wrist

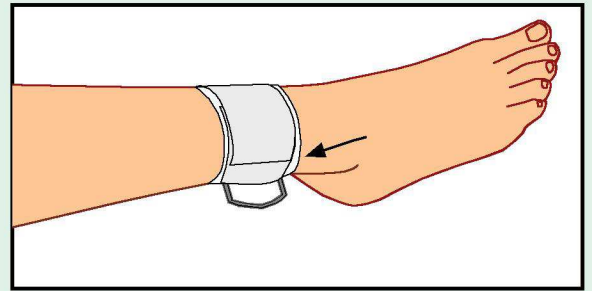

Close the velcro

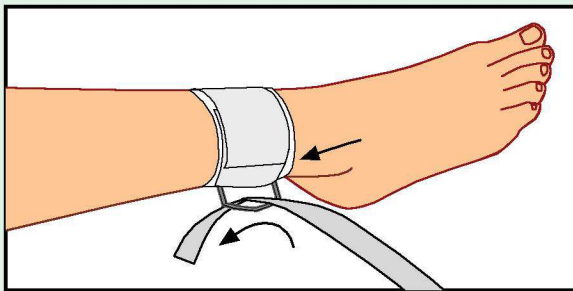

Stick the ribbon through the metal ring

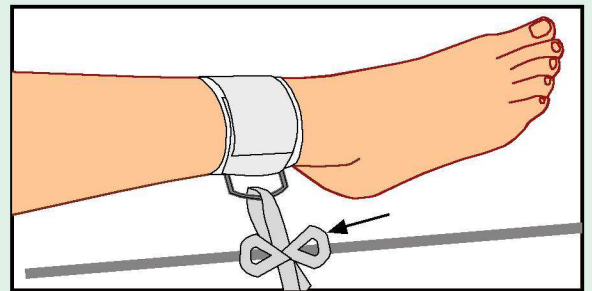

Connect the ribbon to a fixed point



▷ **STERILE**

▷ **Single use**

▷ *Do not resterilize*


▷ *Do not use opened and damaged packs*

USAGE INSTRUCTIONS


FESTERMED METAL®

LIMB HOLDER FOR ADULTS


MODEL : HAND
REF. NO : ALH 2011

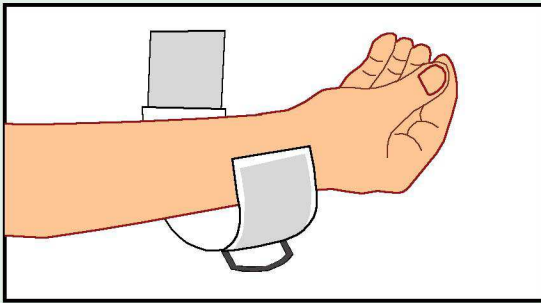


Wrap the band around ankle

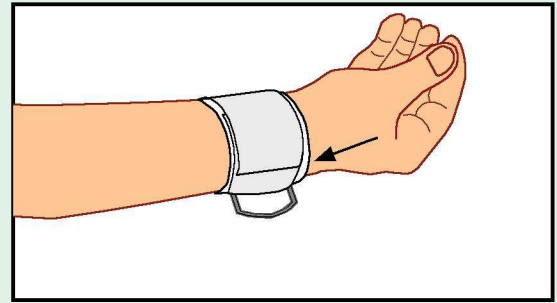

Pay attention for not to hinder the blood circulation

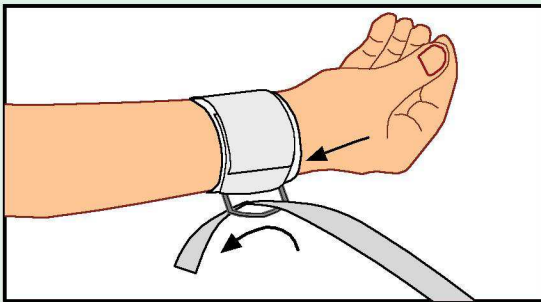

Close the velcro

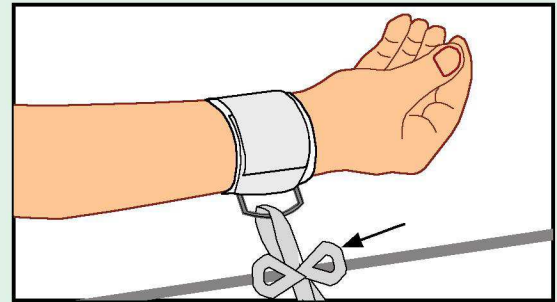

Connect the ribbon to a fixed point



▷ **STERILE**

▷ **Single use**

▷ *Do not resterilize*


▷ *Do not use opened and damaged packs*

USAGE INSTRUCTIONS


FESTERMED

ADULT HAND STABILIZATION TAPE


MODEL: QRB
REF. NO :QRB 2013

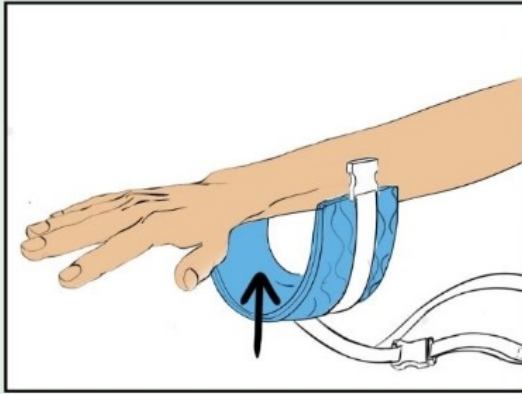


Put the band around the wrist.

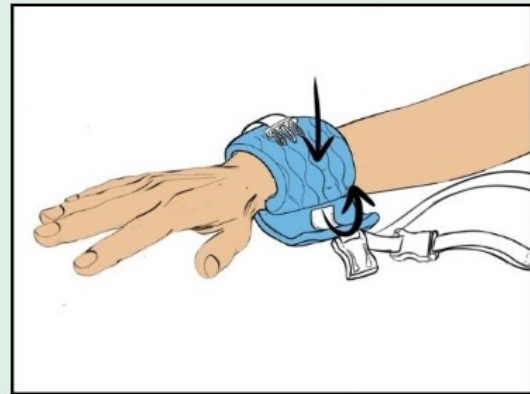

Wrap around the wrist so as not to obstruct blood circulation.

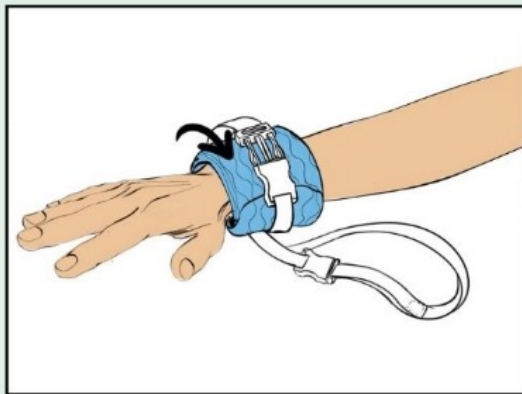


Close the velcro lock.

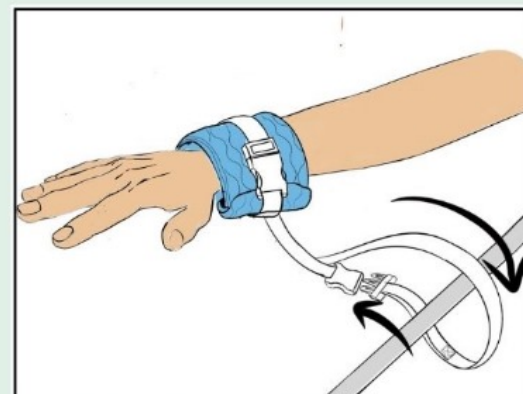

Limit movement by tying the strap of the band to a fixed point.



▷ Non-steril

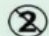▷  Single Use

▷ Do not use opened and damaged packs