


USAGE INSTRUCTIONS


MITTMED

FINGER CONTROL MITT NEW BORN


MODEL : STANDART
REF. NO : MITTP01/MITTP02/MITTP03

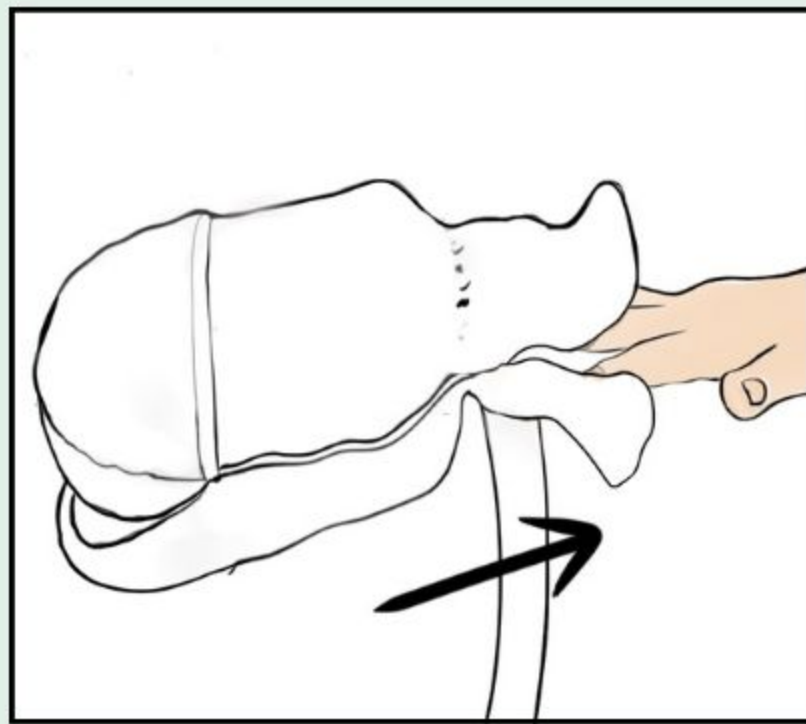


Put the mitt on the patient's hand.

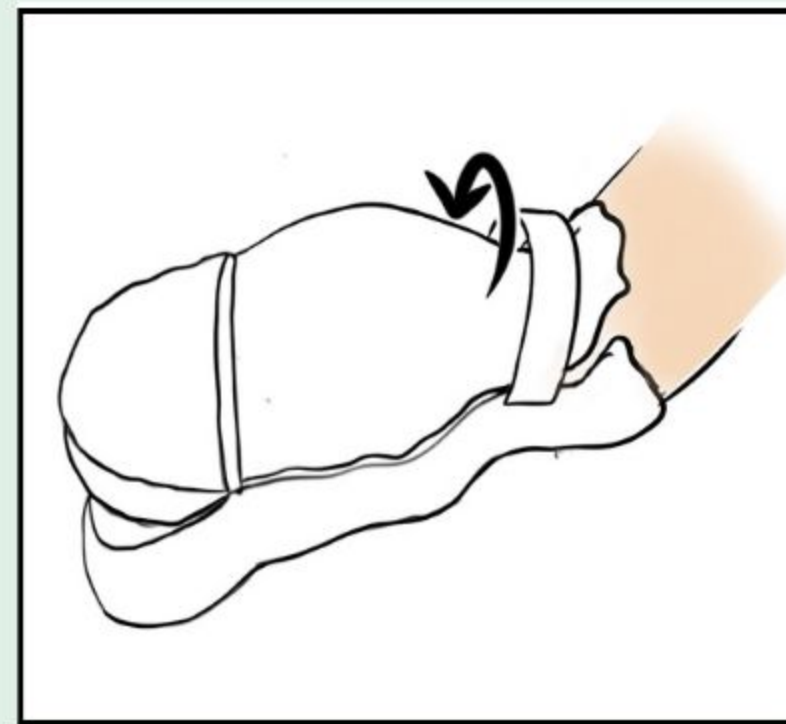

Thread the fingers into the open section.

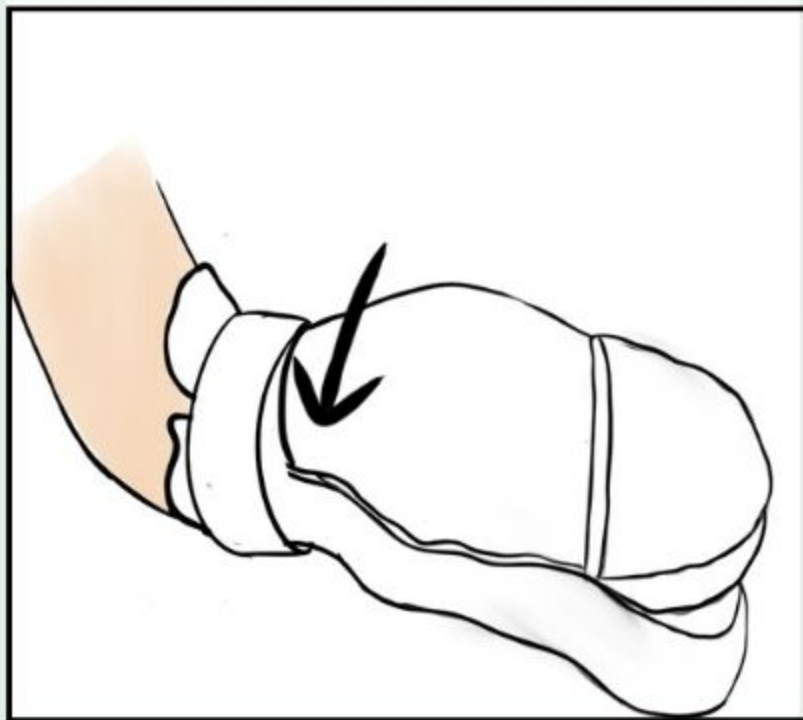


Wrap the band around the wrist through the wrist clasp and secure.

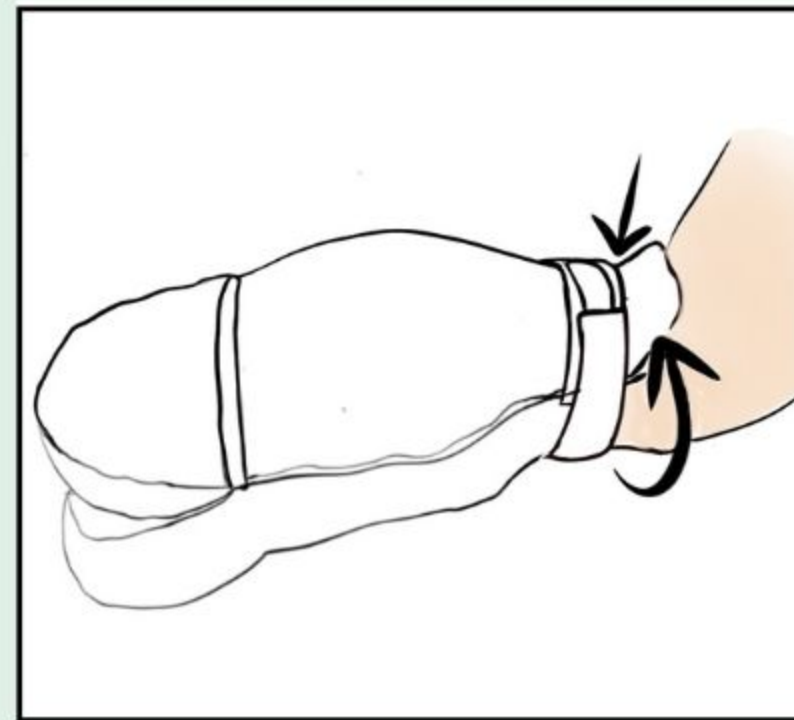

Check that the band does not obstruct blood circulation.



▷ Non-steril

▷  Single Use

▷ Washable at 30°C


▷ Do not use opened and damaged packs

USAGE INSTRUCTIONS


MITTMED

CLOSED FINGER CONTROL MITT


MODEL :CLOSE
REF. NO :MITTCTW

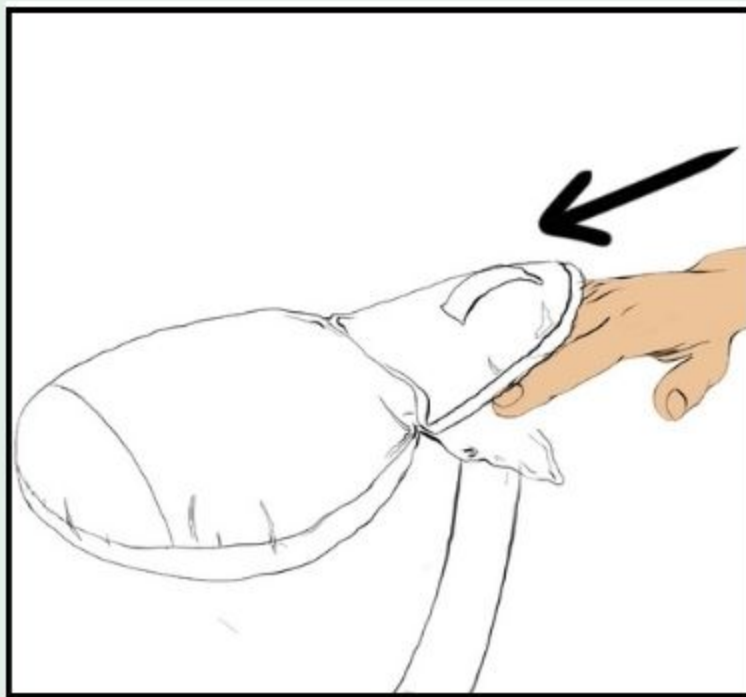


Put the mitt on the patient's hand.

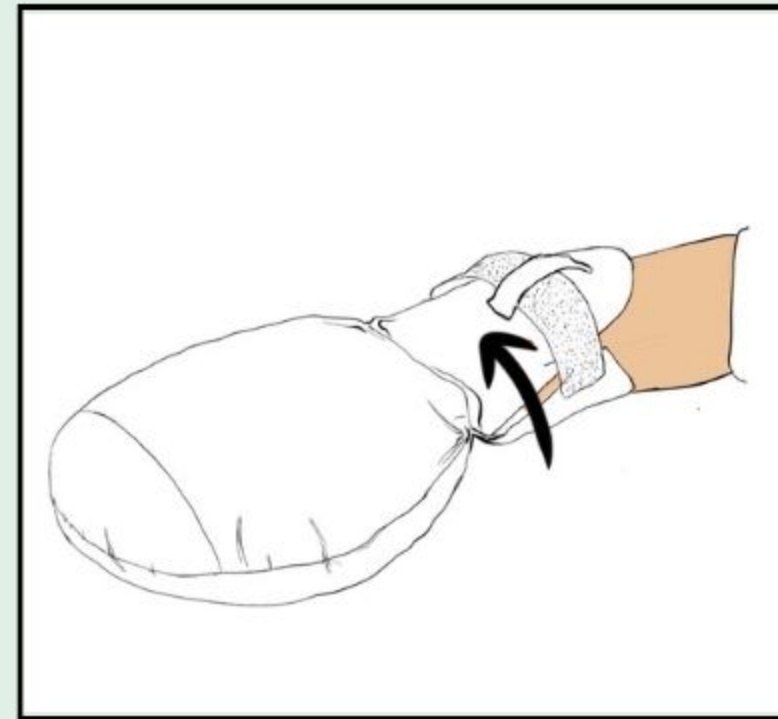

Pass the band through the wrist buckle.

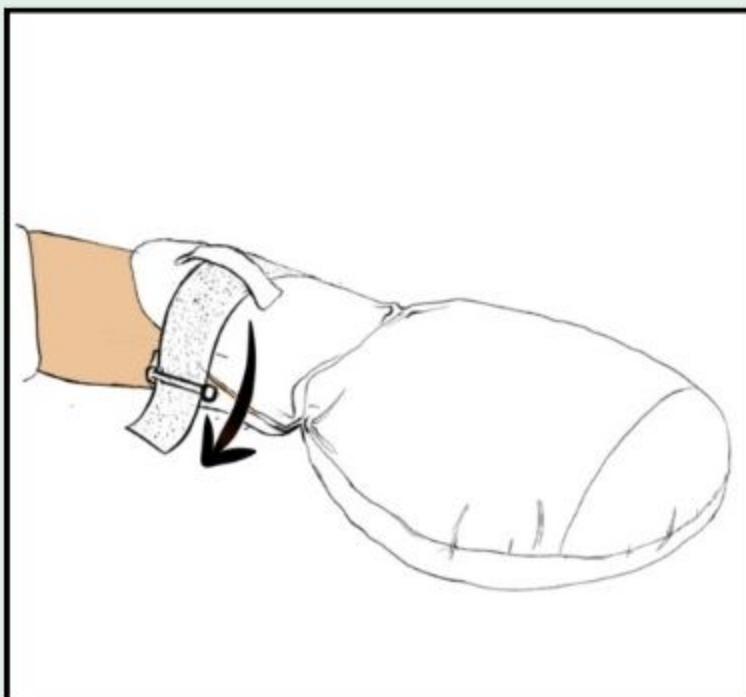


Wrap the band around the wrist and secure with velcro.

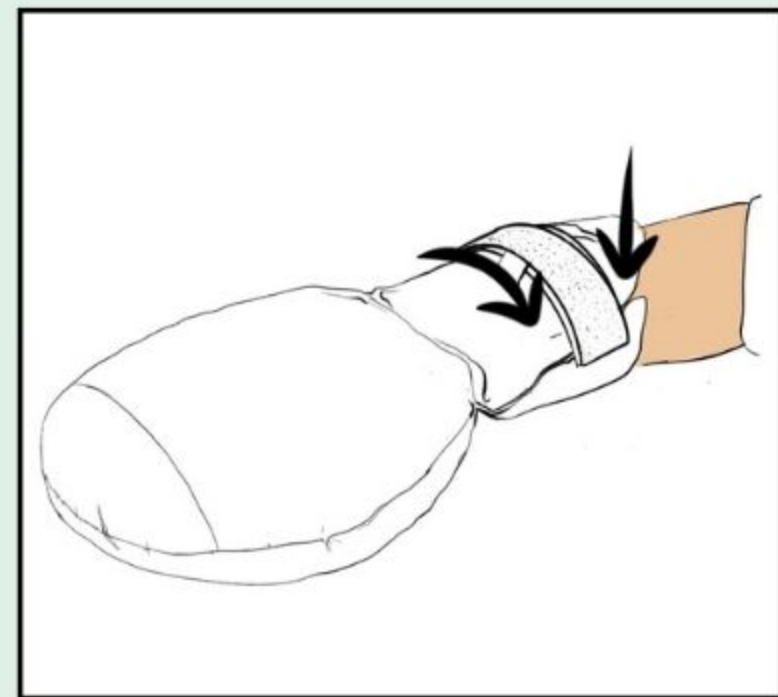

Check that the band does not obstruct blood circulation.



▷ Non-steril

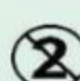▷  Single Use

▷ Washable at 30°C


▷ Do not use opened and damaged packs

USAGE INSTRUCTIONS


MITTMED

OPEN FINGER CONTROL MITT


MODEL :OPEN
REF. NO :MITTOTW

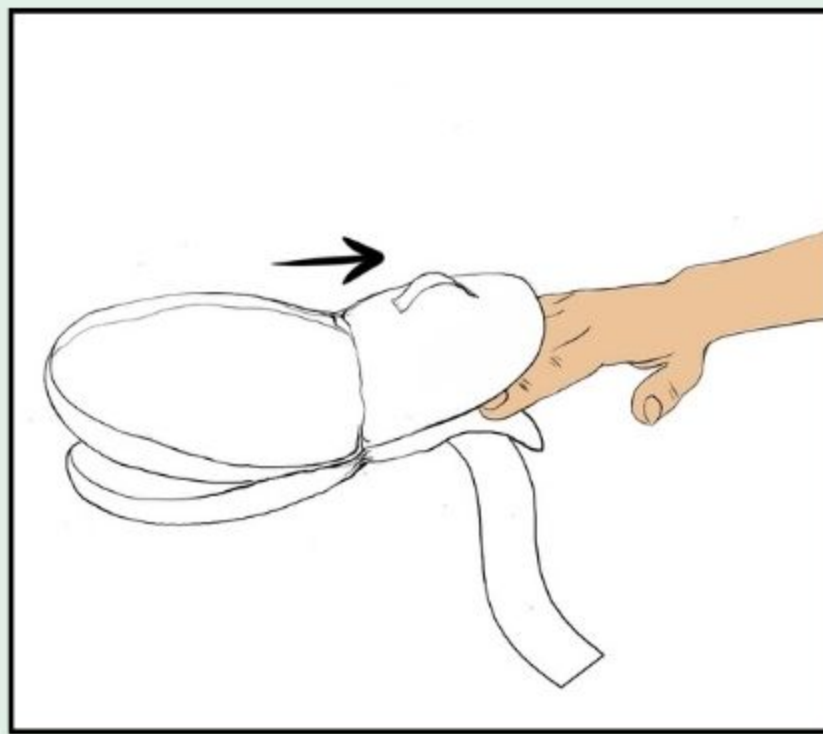


Put the mitt on the patient's hand

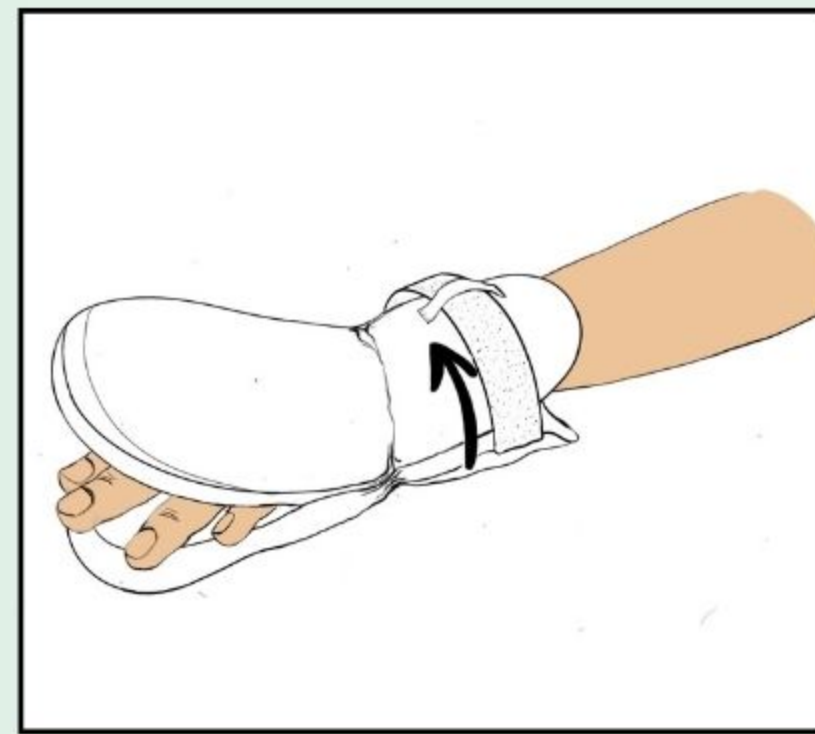

Pass the tape through the wrist buckle.

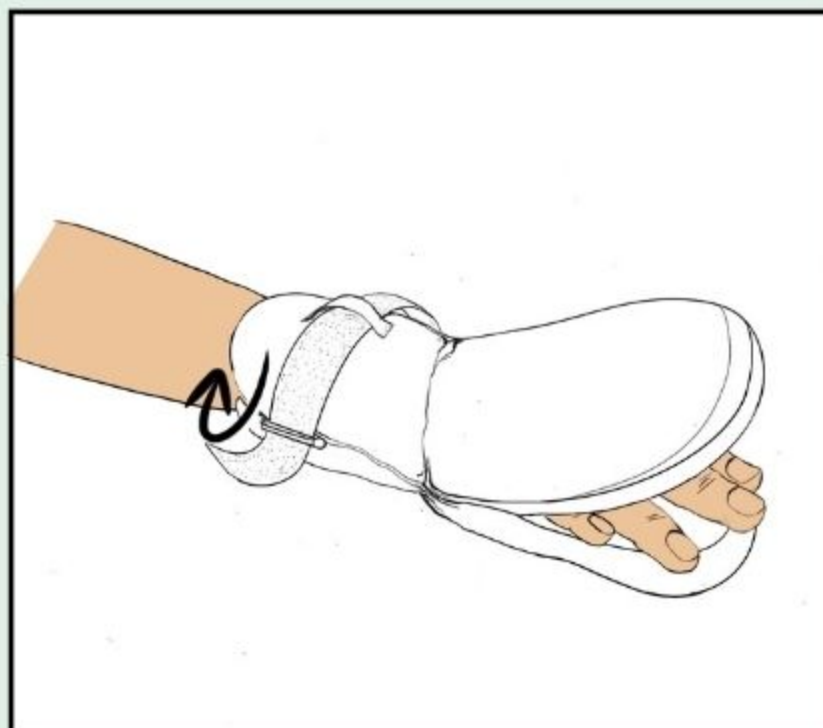

Wrap the band around the wrist and secure

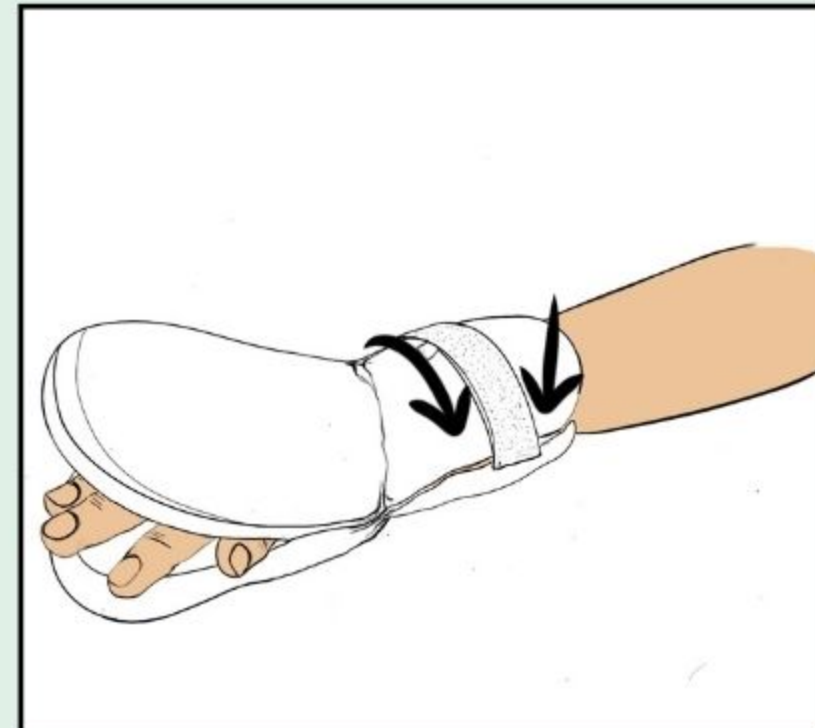

Check that the band does not obstruct blood circulation.



▷ Non-steril

▷  Single Use

▷ Washable at 30°C


▷ Do not use opened and damaged packs

USAGE INSTRUCTIONS


MITTMED

THERAPY MITT


MODEL : THERAPY
REF. NO :MITTSRVTW

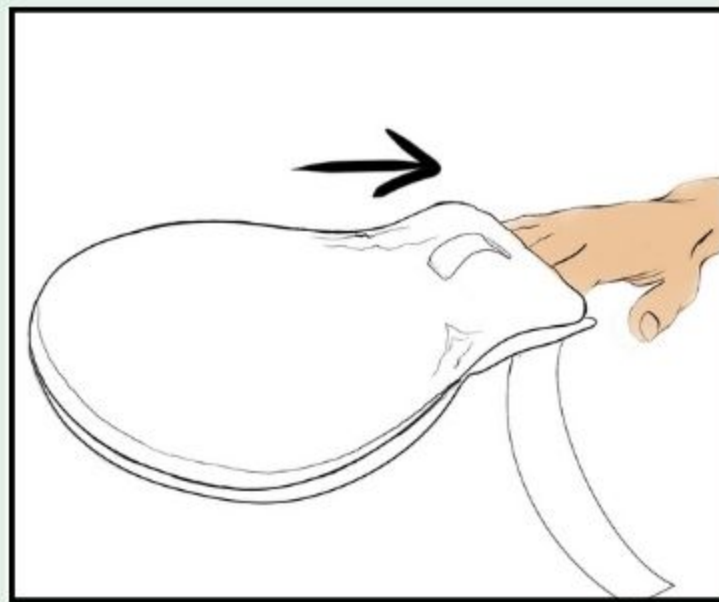


Put the glove on the patient's hand.

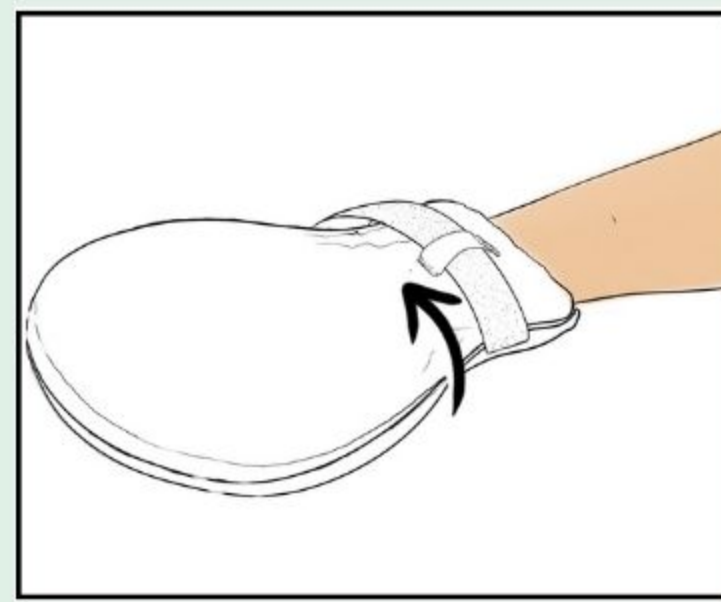

Pass the tape through the wrist buckle.

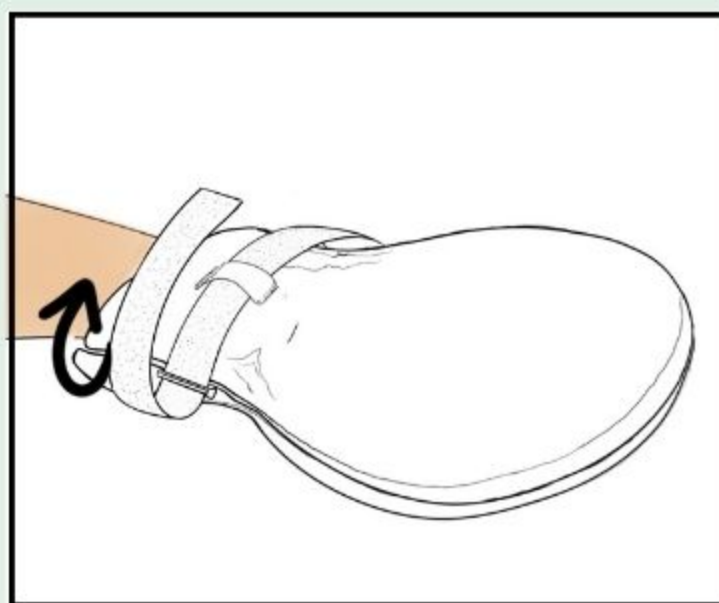

*Pass the tape through the buckle.
Wrap the band around the patient's wrist. Secure the band with Velcro.*

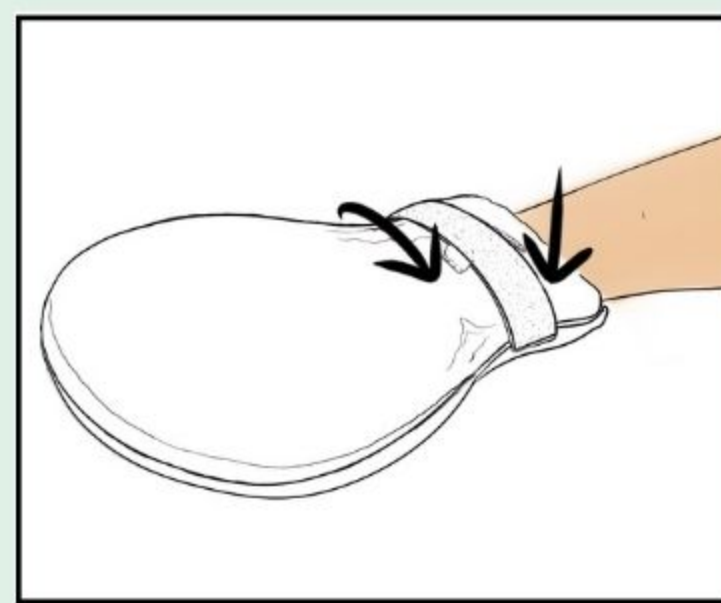

Check that the band does not obstruct blood circulation.



▷ Non-steril

▷  Single Use

▷ Washable at 30°C


▷ Do not use opened and damaged packs

USAGE INSTRUCTIONS


MITTMED

FISHNET FINGER CONTROL MITT


MODEL: FISHNET
REF. NO : MITTFC101

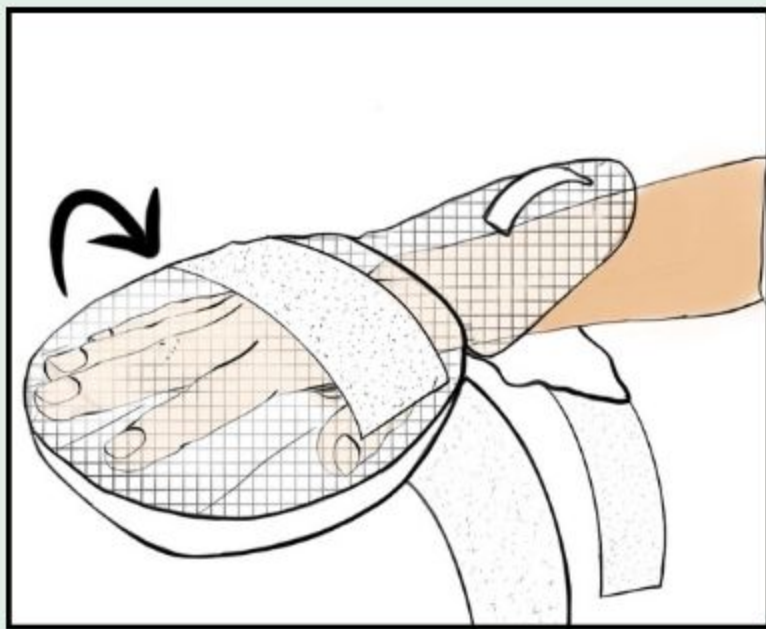


Insert each of the fingers into glove.

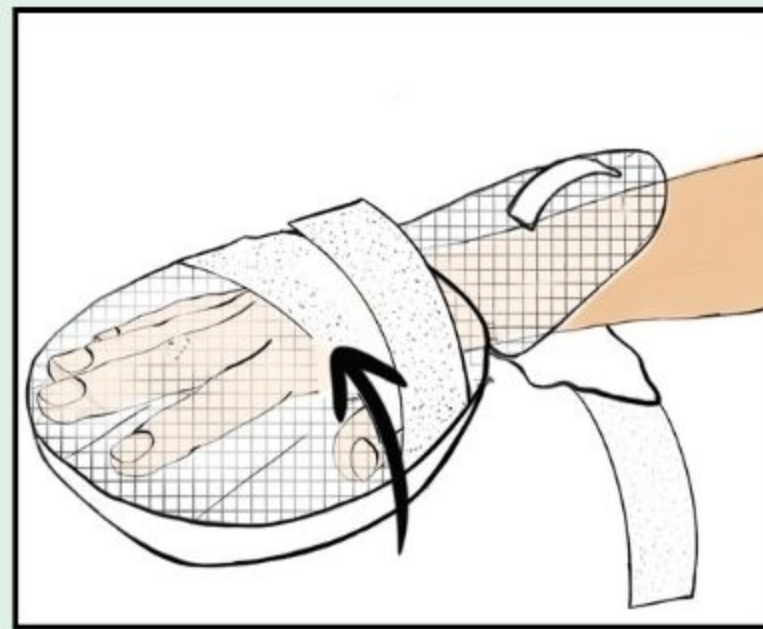


Insert each of the fingers into glove.

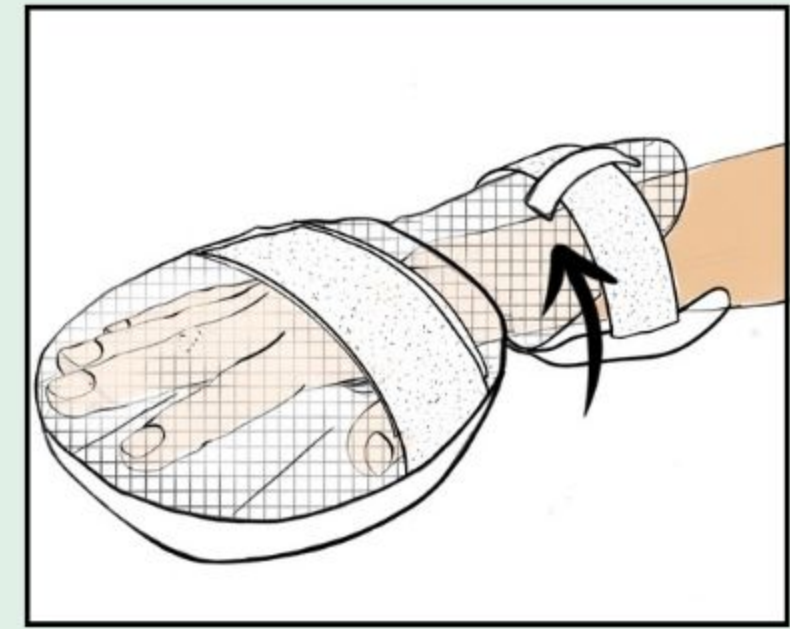


Pass the tape through the wrist buckle.

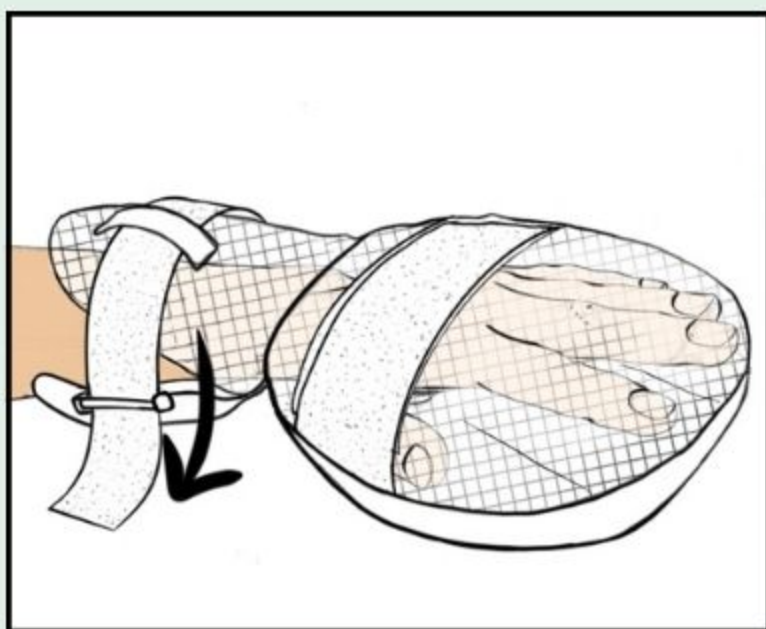

Secure the tape with Velcro

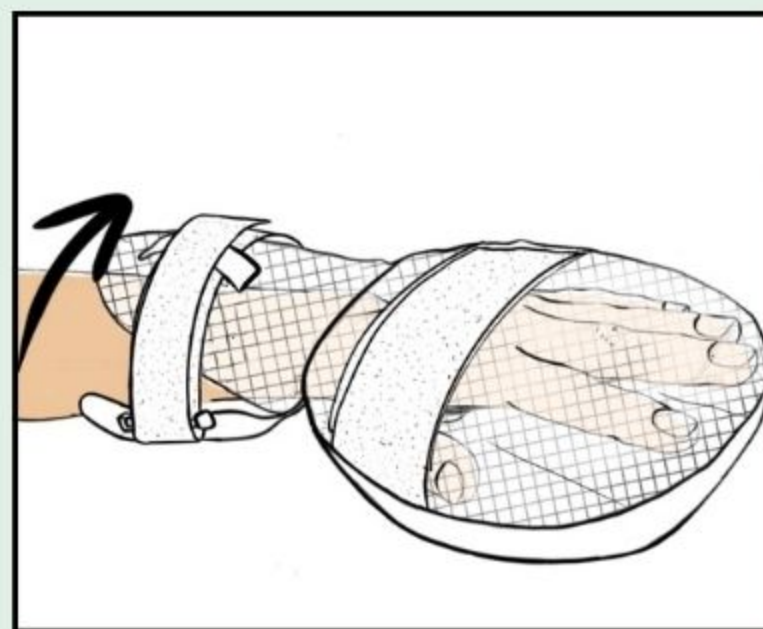


Secure the mitt to the patient's wrist with tape.

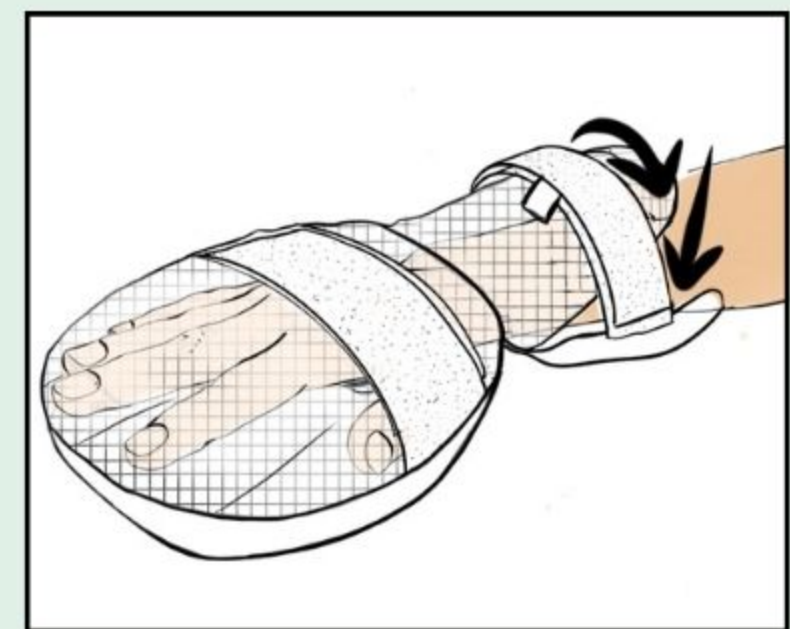

Check that the band does not obstruct blood circulation.



▷ Non-steril

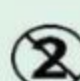▷  Single Use

▷ Washable at 30°C

▷ Do not use opened and damaged packs