


# USAGE INSTRUCTIONS

## **NASOMED TP**


### **NASAL CANNULA FIXATION BAND**

MODEL : CHILD


REF. NO : NTH 5001 TP

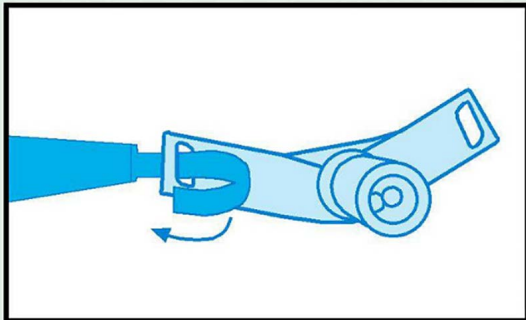


*Put velcro tips of the bands through the holes of the catheter*

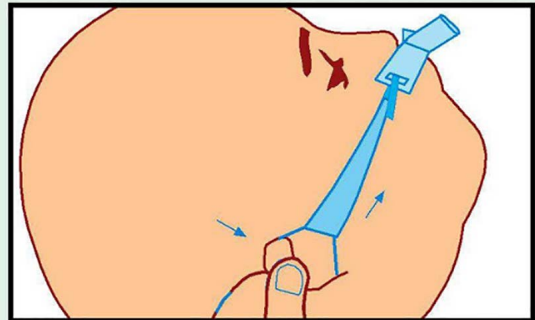

*Place the elastic rings over both ears and adjust the length*

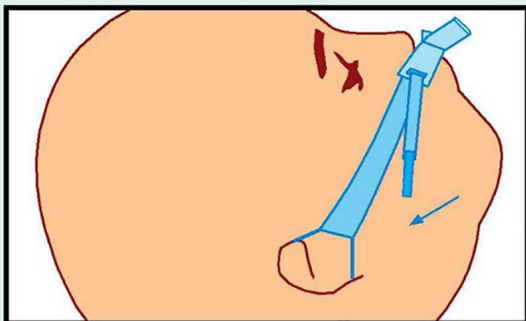

*Close the velcros*

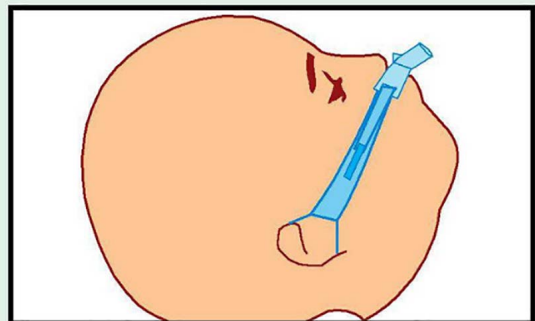

*Be sure that the band holds the catheter safe and stable*



▷ **STERILE**

▷ Single use

▷ Do not resterilize


▷ Do not use opened and damaged packs

# USAGE INSTRUCTIONS


**NASOMED**

## NASAL CANNULA FIXATION BAND


MODEL : ADULT  
REF. NO : NTH 5001

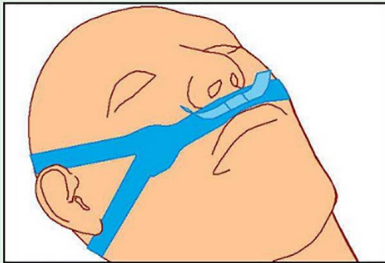


*Place the narrow part of the nasal band onto the upper lip*

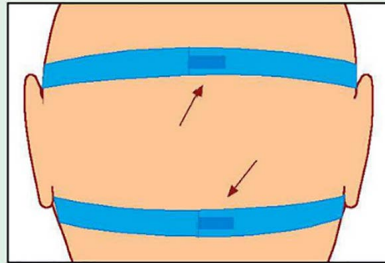


*Pull the ends backwards and close the velcros*

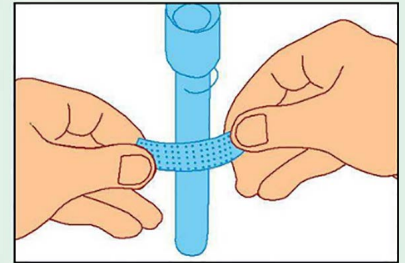


*Stick the adhesive side of the velcro strip on the tube*

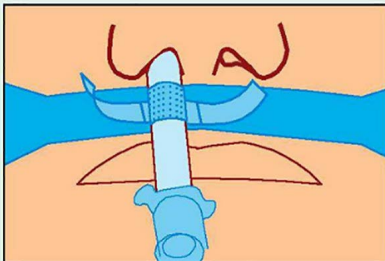

*Pull the nasal tube on the velcro*

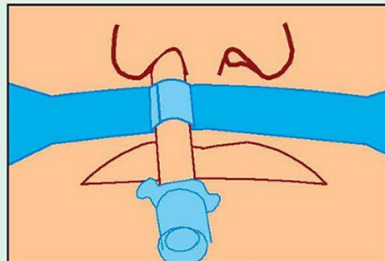

*Wrap the velcro on the band around the velcro on the tube*

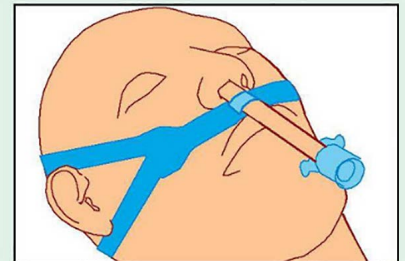

*Check the position of the tube*



▷ **STERILE**

▷ Single use

▷ Do not resterilize


▷ Do not use opened and damaged packs

# USAGE INSTRUCTIONS


**NASOMED**

## NASAL CANNULA FIXATION BAND


MODEL : BABY REF.  
NO : NTH 5001

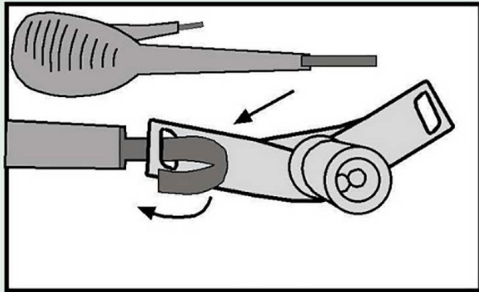


*Put velcro tip through the hole of the catheter*

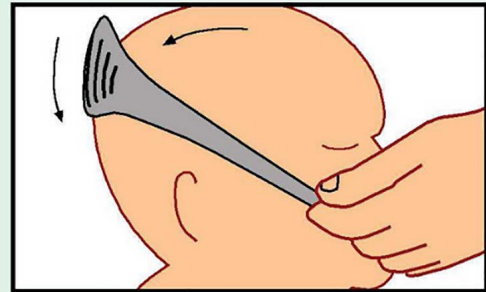

*Wear the bracer over the head and put next velcro tip through the other hole of the catheter*

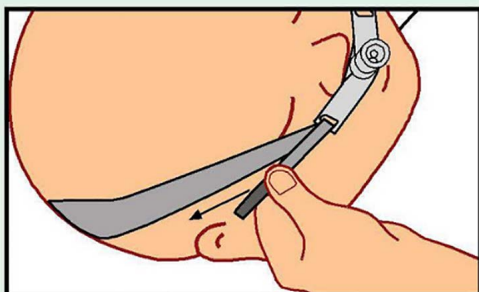

*Place the catheter to the nose and adjust the length and strength of the band*

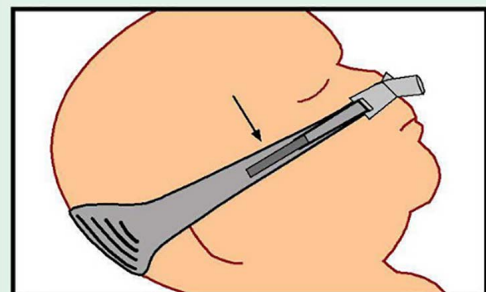

*Close the velcro. Be sure that the band holds the catheter safe and stable*



▷ **STERILE**

▷ Single use

▷ Do not resterilize

▷ Do not use opened and damaged packs